

Pharmtech

BEAUTY INNOVATION

ELAXEN HA-PN

**Dual - Action
Formula**



Hydration perfected, regeneration activated
Confidence delivered

Dual-action performance- hydration + Tissue repair
Longer - lasting effect, less irritation




ELAXEN HA-PN is a pre-filled skin booster combining Hyaluronic Acid and Polynucleotide (PN) to Hydrate instantly and renew skin deeply. it delivers smooth, radiant, elastic skin that looks naturally refreshed.

Dual - Action Hyro - Regenerative Technology :

Hyaluronic acid's role:
Hydration : HA attracts and binds moisture, reviving skin's elasticity and suppleness



Polynucleotide's role :
Regeneration : PN Stimulates fibroblast activity, helping rebuild collagen and improve skin structure

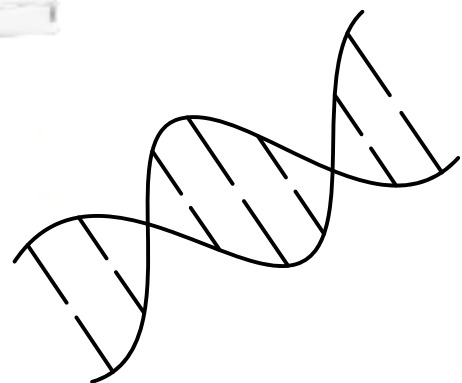

Combination Result :

Balance : Together, they form a Hydro-Regenerative complex that **hydrates, heals, and strengthens skin from within**

Benefits of Polynucleotides (PN) for skin rejuvetaion




Improves
healing and skin
regeneration



Increases
hydration and
mositure
retention in the
skin



Enhances
collagen
synthesis and
skin elasticity

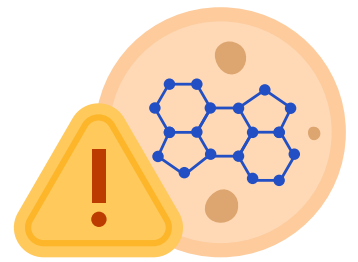

Reduces of
pigmentation
and scars

RESULT



CLINICAL ADVANTAGE

Non crosslinked , non animal ingredients provide superior safety and tissue integration

Pre - Filled sterile syringes
guarantee accurate dosing and zero contamination

— [**smooth injection flow**
ensures patient comfort
and procedural ease

— [Predictable, **natural-looking results** encourage patient
loyalty and repeat visits.

— [**High ROI potential for clinics**
due to minimal downtime and
strong satisfaction rates



Treatment Protocol

Primary Course: 4–6 sessions, spaced every 10 days

Maintenance: 1–2 sessions monthly to sustain results

Injection Depth: Superficial to mid-dermis (microinjection pattern)

Needle Type: 30G × 4 mm or 31G × 8 mm
Volume per Point: 0.02–0.03 mL (1 cm grid spacing)

Treatment Areas: Face, neck, décolleté, hands



Comparative Analysis

Feature	Elaxen HA-PN	Ordinary HA Skin Boosters
Composition	HA-PN Dual complex	Singel HA component
Effect	Hydration + Regeneration	Temporary Hydration only
Longevity	PN- Stimulated Collagen renewal	Short-term
Form	Pre Filled syringe, ready to inject	Manual preparation
Skin Response	Minimal down time for skin	Surface- level mositure only
Clinical Benefit	Higher profitability for clinics	Basic Hydration service

Specifications

Product Name: Elaxen HA-PN

Composition: Aqua (Water), Sodium Hyaluronate, Sodium Chloride, Disodium Phosphate, Sodium Phosphate, Sodium DNA

Package: 2.5 mL * 10 syringes

Duration: Recommended treatment cycle of 4–6 sessions with follow-up maintenance every 4–6 weeks for sustained results.

Storage: Store in a cool, dry place (2–25°C), protected from direct sunlight and freezing. Shelf Life: 36 months from the date of manufacture. The expiry date is marked on each package.



Pharmtech

BEAUTY INNOVATION

 [pharmtech.beauty.innovation](https://www.instagram.com/pharmtech.beauty.innovation)